

**Instruction for Use: Disposable Gowns**

Intended Use: This device are used for to protect both the patient and the operating room personnel from transfer of microorganisms, body fluids, and particulate materialbduring surgical procedures Device is a sterile device and sterilized by method of EO.

Contraindications: NONE-KNOWN

Serility: Euroset Surgical Gowns are sterilizedby ethylene oxide. Do not use if package opened or damaged.

Safety Tips:

- Surgical gowns designed different types acc. to operation characteristics as below;











TYPE OF THE GOWNS	PERFORMANCE LEVEL ACC.TO TSENISO 13795:2013	USED FOR
STANDARD SURGICAL GOWN	STANDARD PERFORMANCE	OPERATIONS W/ LOW LIQUID RISK
REINFORCED SURGICAL GOWN	HIGH PERFORMANCE	OPERATIONS W/ MODARATE LIQUID RISK
FULLY REINFORCED GOWN	HIGH PERFORMANCE	OPERATIONS W/ HIGH LIQUID RISK

Please choose the type of gowns acc.to liquid risk of the operation.

- There are some chemical EO process indicators on the sterilization package. They checked the colour change is suitable or not. If the colour not changes same as indicated on the pouches. Dont use this package.
- Sterile pouches made from medical grade paper and peelableplasic film. Please open it acc. to aseptic technique.
- Surgical gowns should be wear acc. to aseptic rules. If surgeon and /or circulating nurse touch on the outside of the gown, the sterility may broken. In this case this gown should be changed with a new one.
- If any holes and tears on the gowns, dont use it and dispose.
- If there is any damage of the sterile device package, the sterility may broken. In this case, devices which have damaged package should be destroyed.
- If device expired it's shelf life, device should not used.
- Device shelf life stability study is tested with an ambient temprature as 37 deg C. If device storage over 37 Deg. C , the sterility may broken.

Storage: Recommended storage conditions below 37 Deg. C. Do not use after expiry date.

USED SYMBOLS ON THE PRODUCT LABEL FOR SAFETY:

	SINGLE USE ONLY		DO NOT USE DAMAGED PACKAGE
	MANUFACTURER NAME AND ADDRESS		MANUFACTURING DATE
	SERIAL NUMBER		EXPIRY DATE
	CATALOGUE NUMBER		CE LOGO
	STERILIZED BY ETHYLENE OXIDE		MAX. STORAGE TEMPERATURE

NUREL MEDİKAL SAN VE TİC A.Ş.

Demirtaş Dumlupınarosb Mah. Karanfil Sk. No:4 Osmangazi-BURSA/TURKIYE

Tel: +90 224 261 10 20 – Fax: +90 224 261 60 62

www.nurelmedikal.com – info@nurelmedikal.com

**Instruction for Use: Disposable Surgical Drapes**

Intended Use: This device are used for barrier to liquid and microbial penetration between patient and operator room, the other surgical devices, operation room personnel. Devices are sterilized by method of EO and devices are sterile devices.

Contraindications: NONE-KNOWN

Serility: Euroset Surgical Drapes are sterilized by ethylene oxide. Do not use if package opened or damaged.

Safety Tips:

- Surgical drapes designed different types acc. to operation characteristics as below;

TYPE OF THE DRAPE	PERFORMANCE LEVEL ACC.TO TSENISO 13795:2013	USED FOR
STANDARD PERFORMANCE DRAPES	STANDARD PERFORMANCE	OPERATIONS W/ LOW-MODARATE LIQUID RISK
HIGH PERFORMANCE DRAPES	HIGH PERFORMANCE	OPERATIONS W/ HIGH LIQUID RISK

Drapes are designed acc.to liquid risk level of the operation.

- There are some chemical EO process indicators on the sterilization package. They checked the colour change is suitable or not. If the colour not changes same as indicated on the pouches. Dont use this package.
- Sterile pouches made from medical grade paper and peelableplastic film. Please open it acc. to aseptic technique.
- Surgical gowns should be packaged to aseptic rules.
- If there is any damage of the sterile device package, the sterility may broken. In this case, devices which have damaged package should be destroyed.
- If device expired it's shelf life, device should not used.
- Device shelf life stability study is tested with an ambient temperature as 37 deg C. If device storage over 37 Deg. C , the sterility may broken.

NUREL MEDIKAL SAN VE TIC A.Ş.








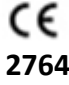


Demirtaş Dumlupınarosb Mah. Karanfil Sk. No:4 Osmangazi-BURSA/TURKIYE

Tel: +90 224 261 10 20 – Fax: +90 224 261 60 62

www.nurelmedikal.com – info@nurelmedikal.com

Storage: Recommended storage conditions below 37 Deg. C. Do not use after expiry date.

USED SYMBOLS ON THE PRODUCT LABEL FOR SAFETY:

	SINGLE USE ONLY		DO NOT USE DAMAGED PACKAGE
	MANUFACTURER NAME AND ADRESS		MANUFACTURING DATE
	SERIAL NUMBER		EXPRIY DATE
	CATALOGUE NUMBER		CE LOGO
	STERILIZED BY ETHYLENE OXIDE		MAX. STORAGE TEMPERATURE