


COMPACT

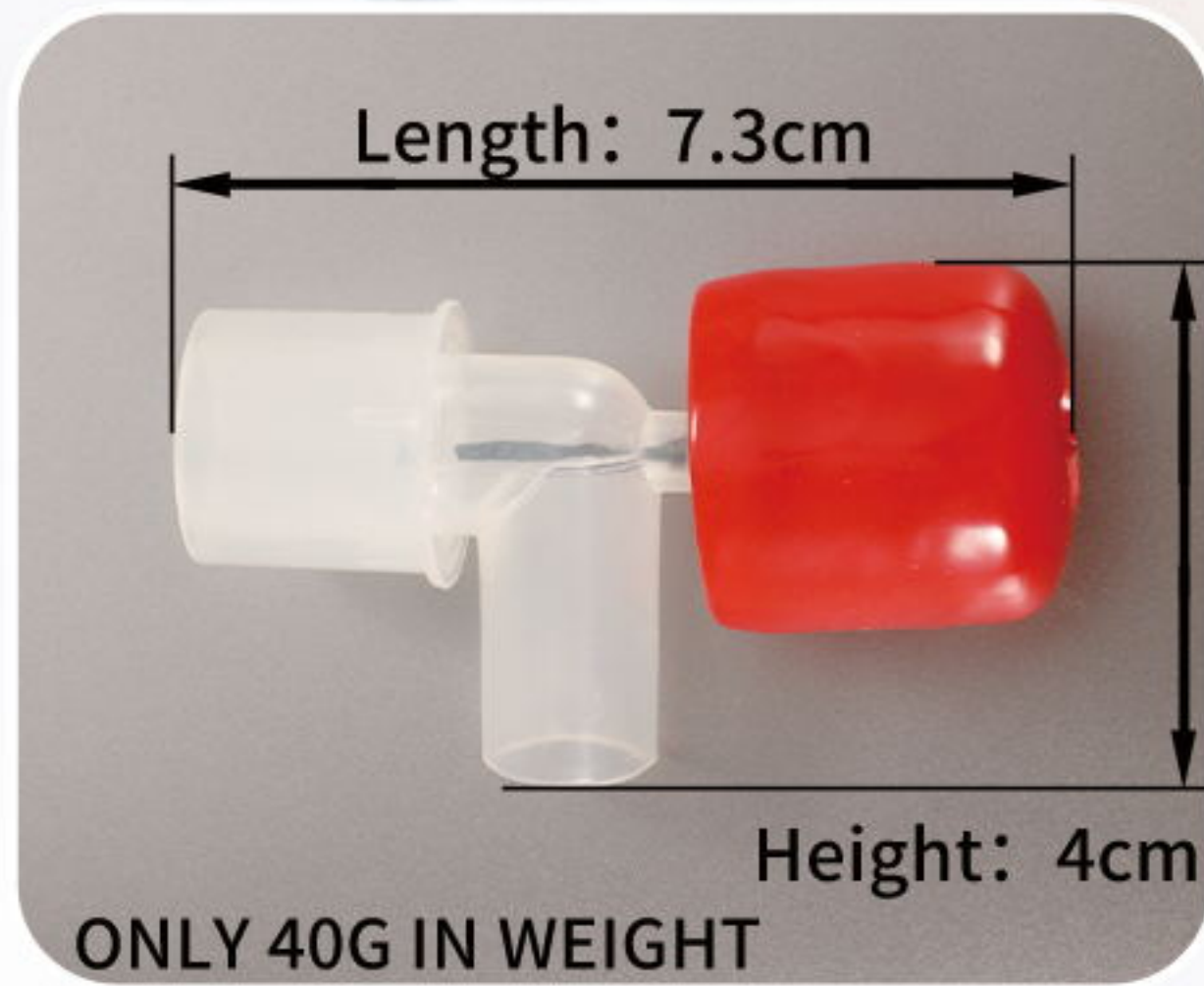

EASY TO USE

Only connect the Pet Apnea Monitor between the endotracheal tube and your anesthesia machine, then you are set to go!

SENSITIVE TO APNEA

If your pet fails to breathe for 30 seconds or more, Pet Apnea Monitor will start to sound beep.

TO TURN ON AUTOMATICALLY

Just connect with the pet. The first breath of your pet will turn on the Pet Apnea Monitor automatically!

TO TURN OFF AUTOMATICALLY

When the pet is disconnected, the Pet Apnea Monitor will wait 25 seconds, then beep for 20 seconds before turning off.

BUILT-IN BATTERY

Pet Apnea Monitor adopts a long life Lithium battery to provide up to 2 years of service. No battery changes or maintenance.

EASY TO CLEAN

Use a cloth dampened with isopropyl alcohol 70% to wipe it, then use a clean cloth to dry it before use. Do not immerse it in liquid or sterilize it with autoclave.

TO KEEP IT IN AN ENCLOSED SPACE

When you don't use it, please keep the Pet Apnea Monitor in plastic package or a drawer to avoid triggering its sensitive function.

ONE YEAR WARRANTY



(Pet Apnea Monitor is used to monitor the apnea of a 2-months cat (about 425g) under anesthesia)

Note: Almost all veterinary anesthetic drugs depress respiration in a dose related manner. This makes respiration monitoring an ideal tool to assess the anesthetic depth. Most emergencies during the anesthesia can be attributed to human errors, simply due to lack of vigilance. The Pet Apnea Monitor is always vigilant, not only preventing deaths but allowing better management of every anesthetic.