

Thank you for purchasing Meditech Pet Heating Pad

Meditech Pet Heating Pad is designed to provide pets with adjustable 86~121F even and constant temperature, ideal for all pets in cold winter looking for warm spot to snugg, aging pets or those with arthritis or rheumatism, new born puppies and kittens etc, really your pets best mate in cold winter. Meditech pet heating pad is designed for use with animals only.

WARNING: This is an electrical product. Read all directions and precautions carefully before use. Failure to follow all instructions may cause electrical shock or fire which could cause serious bodily injury.

Important safeguards

When using electrical appliances, basic safety precautions should always be followed including the following:


- 1 Read all instructions before use.
- 2.To protect against electrical hazards, DO NOT immerse cord, plug, or appliance in water or other liquid.
- 3.Not for use on humans.
- 4.Use only with supplemental cover supplied with
- 5.Do not wash this heat mat.
- 6.Do not machine wash. Spot clean by hand only.
- 7.Discard this pad if it shows any signs of deterioration to the pad or its cord.
- 8.Closely supervise children near the heated mat.
- 9.Unplug when not in use!
- 10.Do not puncture pad for any reason.
- 11.For use with provided power supply only, do not modify electric connection in any way.
- 12.When placing the pad in an enclosed area such as a dog house or dome, be sure the pad covers no more than half the floor space so your pet is able to sleep on or off the heating pad.


INSTRUCTIONS

- 1.Place directly on a flat surface indoor, make sure no any sharp or hard object against the heating pad.
- 2 Make sure the power cord is properly handled if you install this pet heating pad in a dog bed, dome or house.
- 3.Make sure the fleece cover coming with heating pad is on heating pad before using it with you pets. Do not let pets contact the heating pad directly without the fleece cover.
- 4.Carefully check the conditions of the heating pad if any sign of damage before you plug in.
- 5.Plug into 110-120 volt AC outlet, for fast heating you can set temperature at the highest level 50C/121F at first 30 minutes, sometimes pad will feel slightly warmer than the ambient temperature but when pet lies on, heat can't lose, it will reach the set temperature and stay at that temperature evenly and constantly.
- 6.The Meditech pet heating pad has bi-metal thermostats built inside, can control the temperature automatically and prevent mat from over-heating.
- 7.Unplug when not in use, clean with damp cloth and store properly.

Controller Use Guide:

*On/off: Long press  key for 2 seconds to turn on or turn off the controller.

*Timer: When controller turns on, press  key to set timer from 2 hours to 24 hours(Always on), default timer is 2H.

*Temperature:When controller turns on. press  key to set up temperature from 30C/86F to 50C/121F, Default temperature is 40C/104F.



Note: If you intend to use this pet heating pad for more than 4 hours, we suggest to set temperature at 40C/104F which is vet recommended temperature for pet body.

Specification Chart

Size	Model	Rating	Dimension
S	THC-3845	120V 60Hz 25W	15.75*18``
M	THC-4371	120V 60Hz 45W	17.7*28.7``
L	THC-5080	120V 60Hz 45W	19.7*31.5``
MADE IN CHINA			

Warranty

1. One Year Limited Warranty
2. Should you have any questions during the use of Heatideas Pet Heating Pad, please feel free to ask for assistance at support@thermal-ideas.com. We'd love to be in contact with our customers and offer excellent aftersale service.

Heatideas pet heating pad, cradle your pets in cozy warm!