

Pet Scale TTPS-20

WARNING

Do not expose the scales to moisture
Do not drop or mistreat the scales
Do not immerse the scales in water
Do not expose the scales to extreme heat
Do not overload the scales-this will permanently damage them
Do not leave weight on the scales when not in use
Never leave baby unattended when using the scales
Replace batteries when the scales display LO

For using and maintaining the baby scales, please read these instructions completely before operating

Specifications

These scales are equipped with a high precision "strain gauge" sensor and a high performance CPU (Central Processing Unit) to ensure weighing accuracy

- Weighing Capacity: 20kg
- Display Resolution: 10g
- Tare Function
- Auto Power Off
- Low Voltage Indicator
- Power Source: 2x AAA Batteries
- Operating Temperature Range: 0-40 degree
- Display: LCD Display
- 150 cm soft measuring tape

Key Functions

ON OFF/TARE key: Power on, Power off, Tare

UNIT key: — Select the units of measurement: kg/lb/oz

- Long press this button, convert auto lock function and unlock function

Operation

Remove the scales from the packaging, open the battery cover and install the batteries. Put the scale on a flat table, making sure that it is stable.

Operating conditions : Room temperature: 10 ° C - 40 ° C. Humidity: 30%-75%. Pressure: 700 - 1060 hPa.

Transport and storage conditions : Temperature: 10 ° C - 40 ° C. Humidity: 5%-95%. Pressure: 700 - 1060 hPa.

Weighing your Baby

1 Press the [ON] key to turn on the power, followed by "0.00."

2 Press the [UNIT] key to select between kg, lb and oz

3 Put your baby on the scales: the LCD display will show the baby's weight

4 Press the [OFF] key for 3 seconds to switch off the scale

WARNING: Ensure that the scales stand stable on a flat surface before weighing. Make sure that the baby lies still on the scales.

Using and Installing the Batteries

Use two AAA batteries

Open the battery cover on the back and put the two AAA batteries into the battery holder, making sure the poles are aligned correctly before closing the battery cover firmly again

Advice for Use and Care

- Put the scales on a clean, flat table.
- To ensure long life, do not expose the scales to water or extreme heat and cold.
- This product is a high precision electronic weighing device, do not press hard or drop the scales as this may cause damage.
- If the scales surface is dirty, Clean the scales with a soft cloth: do not use any chemical cleaning agents and do not allow water to penetrate inside the scales.
- If the display value is incorrect, make sure that the batteries are adequately charge and make good contact with the terminals.
- Replace the batteries if they have been in use for a lengthy period of time and try again

Please note: This symbol means that products bearing it may not be disposed of with household waste.

Always contact your local council or authority for information regarding the correct disposal / removal of old appliances, do not dispose of with household waste.

Packaging

All of the packaging supplied with your appliance can be recycled. Or disposed of without any harm to the environment.

Cardboard may be crushed, or cut into smaller pieces if required for recycling or disposal.

The polythene included contain no CFC's, And are environmentally friendly.

For further information on recycling, contact your local council or authority.

Patented products, infringement will be investigated