



Quick Manual of Digital Oral scanner



Software of Digital Oral Scanner

Digital oral endoscopic system



Patient Management



Images Collection



Image Contrast




Image Editing




Report Printing



System Set

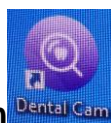

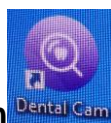Double-click the desktop icon  to start up the software of digital oral scanner.



Add a new case

The screenshot shows the 'Dental Cam' application window. At the top, there are three tabs: 'Add', 'Query', and 'Current'. The 'Add' tab is active. On the left side, there is a vertical sidebar with icons for 'Patients', 'Diagnosis', 'Comparison', 'Edit', 'Report', and 'Settings'. The main area contains a form for adding a new case. It starts with a large 'D' icon representing a patient. To the right of the icon are several input fields: 'Name', 'Age', 'Phone', 'Address', and 'Remark'. There is also a 'Gender' field with radio buttons for 'Male' (selected) and 'Female'. An 'ID card' field is located to the right of the 'Age' field. A 'Save' button is positioned on the right side of the form. At the bottom of the window, there are two sections labeled 'Image' and 'Video', each with a left and right arrow for navigation.

Four split screens for comparison



1. **Patient:** help doctors systematize the management of patient files, key words can query patient information.
2. **Diagnosis:** The main function is “photo” and “video”, the other auxiliary functions are zoom/zoom/actual size/spread /vertical image/horizontal image/ full-screen/video settings, photos and videos taken are automatically saved to a list of images in the lower left corner for comparison and editing.
3. **Comparison:** support real-time shooting in comparison interface, four pictures for comparison, and images can be saved directly to the computer or click on the QR code for transmission.
4. **Edit:** The image can be annotated to facilitate doctors to explain the patient analysis
5. **Report:** Edit case book, add patient information and real time examination picture, fill in examination result and doctor's suggestion, print or transfer.
6. **Settings:** support to choose Chinese or English and set the name of the hospital and title of report or to modify them.



Tootoo Meditech Co., Ltd.

**Address: 3A, Phase II, Smart Home, Baolong Street,
Longgang District, Shenzhen, Guangdong, China**

Tel: +86-755-28306385

Email: info@newtootoo.com

Web: www.newtootoo.com