

Underwater Dog Treadmill

User's Manual



Table of Contents

Warning	2
Introduction ...	3
Product's Using Instruction.....	4
Product's Sport Mode	6

3. Parameters

Voltage: 220v

Lowest Speed: 0.8km/h

High Speed: 12.0km/h

Longest sports time in time model: 99mins

Shortest sports time in time model: 8mins

Longest distance in distance model: 99km

Shortest distance in distance model: 0.8km

Maximum calorie consumption in calorie model: 999cal

5. Sport mode

All the sports model is based on the two premises of A: the treadmill is plug the power cord ; B : the safety key is inserted

1. Manual

- insert the safety lock, Windows display “0.0” ,enter “ start/stop” ; “speed” from the countdown 3 to 1, buzzer warning sound ,system boot up .
- treadmill adjustment the speed as “speed+, speed-”

c. Treadmill count the sports parameters and display updating .

d: press the speed shortcut key adjust the speed directly, , display number the same as key number.

e: press the “stop” to slow down, the treadmill stop.

2. Model

A. Enter: press “mode” the windows flashing display, prompt to choose “time, distance, calorie” model. Press “speed+, speed- ” to adjust the amount of exerciser, press “ start ” to enter the mode, the system start


B. treadmill adjustment the speed as “speed+, speed-”



7. Left Button : Outer water



8. Right Button : Heating, must keep the water have 25cm over the running belt before Heating,

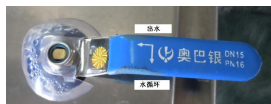

9. Turn up : Outer water ; Turn down : Water cycle



10. Controller for use: Temperature 30-70° C

The controller is an upgraded version of the two digital electric water heater controller, simple operation, easy to use, due to technical upgrading, making the interface more intuitive display, control is more subtle. Specifically described as follows:

Standby: power water heater into standby mode, four digital tube display settings issues and water temperature, continuous pressing the "set temperature" key to adjust the set temperature.

Power status: Press the "ON / OFF" key to enter the water heater work boot, instant heating temperature control. After use press the "On / Off" button, the water heater back into standby.

Timing Status: Timing is set the timer start time, in the standby mode, press the "Timer" button to set the time value, after completion of the countdown, after the return 0, automatic boot.

(Note: Use only the standby timer, you can not use the boot) malfunction:

E1: temperature fault (the water temperature exceeds 90 ° C)

E2: dry fault

E3: sensor open short circuit fault

WARNING: Please use the treadmill on a grounded power outlet.

Please read the instructions carefully before use, and note the following safety precautions.

1. matters placed on the treadmill indoors, away from moisture, water can not be spilled on the treadmill, you can not put any foreign object on the treadmill.

2. Before exercise remember to hang any exerciser strip or ribbon woven cloth, to prevent the movement blew.

3. The power plug must be grounded, outlet dedicated loop, avoid sharing with other electrical equipment.

4. Children should stay away from the machine, to avoid accidents.

5. It should avoid overload ultra-long run, which will cause the motor, controller damage, accelerate bearing, running belt, running wear and aging board regularly do maintenance.

6. reduce indoor dust, keep indoor humidity must avoid static electricity products strong, it may interfere with the normal operation of the electronic form, the controller.

7. sports use, please turn off the treadmill power

8. When using the treadmill, keep the indoor air circulation

9. If you feel there is not suited to the use of animals or unusual circumstances, please stop exercising and consult a doctor.

10. This product must be under the guardianship of people use, to prevent animal damage.

11. can not let the animals freely on a treadmill or playing next to a treadmill.

WARNING: To reduce the following provisions accidents or injuries to others, please observe:

1. Remember before exercise exerciser the treadmill must fill water , and water must be 15cm on the Tempered glass . Do not use without water , otherwise it will broken up the Electric parts
2. Do not give the exerciser with easy hook of any textile fabric.
3. Do not place the power cord near hot objects.
4. Do not let children touch the treadmill
5. Do not use the treadmill outdoors, not on the commercial gym or outdoor using.
6. must cut off the power before moving treadmill.
7. Non-professionals are not allowed to disassemble the unit, otherwise there will be serious consequences may occur.
8. The treadmill only for 63A in the circuit.
9. When the machine starts, only one animal movement on a treadmill.
10. The movement, if found, dizziness, chest pain, nausea, or shortness of breath , stop movement immediately, please consult accordingly trainer or doctor.
11. This treadmill must be under someone's care use, to prevent animal damage.

Introduction

1) Technical Parameters

Working Voltage: AC220V-240V, 50/60HZ

Max. Loading : 80KGS

Product Size: 1580×615×1000mm
Carton Size: 1660×680×900mm
Walking Area L*W: 1300×470mm
Motor Peak Power: 2HP
Speed: 0.8-12km/h

2) Parts List

Parts Name	QTY
Product (base main frame Timer)	1
Tools bag	1
Manual	1

Product Using Instruction:

1. brief introduction

Product must be used according to the instruction manual

Treadmill parameters: speed

Sports parameters: distance, calories, time, speed

2. Display Windows

LCD Display the following functions

1. Speed 2. Time 3. Distance 4. Calories

3. Button:

The treadmill also has another button board, except the main board.

- | | |
|-------------------|-------------------------|
| 1. Start button | 2. Stop/ restart button |
| 3. Program button | 4. Model button |
| 5. Speed+ | 6. Speed - |